

INSTRUCTIONS FOR TAKING EAR IMPRESSIONS

Please pay attention to following aspects for impressions especially for in-ear earphones and ear molds

Dear hearing aid acoustician,

when taking ear impressions for in-ear monitors, please pay attention to the following:

1. Please do not use inviscid materials for molding but such that build up pressure. For our specific use, optimal products include: Otoform A/K (Dreve), Egger A II or Detax Xtra ferm.
2. The ear impressions should reach and slightly pass the second bend or the ear canal. Concha and cymba also have to be filled up completely to guarantee a good reproduction of the ear's anatomy.
3. Please note that we also need enough material to fill up the space around Incisura and Incisura Anterior. This is necessary for a secure fit. We suggest to smooth down the material by applying a flat sheet of foil.
4. Please do not cut, grind or further manipulate the finished ear impressions.
5. Due to high volume on stage, musicians absolutely require a very good seal and, furthermore, a very secure fit - especially for very active performances. Thus we kindly ask you to retake the impressions should you notice any blemishes around the area crucial for a good seal or those for a tight fit.

If you have any questions, encounter problems or notice any abnormalities, please get in contact with us.

Thank you for your cooperation!